



Thermal Controlled Procedure Pad

DIRECTIONS FOR USE

1. The Thermal Controlled Procedure Pad is a sealed electrical device. Before use, inspect the pad to ensure that the surface has not cracked from wear and that the cord is in proper condition. If the surface is cracked, do not use the unit.
2. The pad should be plugged into a 110VAC outlet. The indicator light on the unit should come on. Once the pad reaches the optimum temperature range, the green light may cycle on and off. This is normal operation.

IMPORTANT

*Whenever the pad is plugged in, it will start to heat, however, the thermal mass of the unit will initially lag behind the heater. Therefore, it will heat beyond its operating range in the first cycle. An animal should not be placed on the unit during this initial conditioning period. **Allow the pad to cycle for approximately 30 minutes before placing an animal on the pad.***

3. Before using the pad each day, you must check its temperature to ensure that it is operating properly. To do so, be sure to wait 30 minutes after plugging in the pad. To check the temperature, lay the probe of a shop thermometer on top of the thermal pad and cover it with a towel or cloth. Do not use the pad if the temperature exceeds 103.5°F. **Do not use an infrared thermometer to check the temperature.**

4. To clean the pad, use any household cleaner that is safe for plastics.
5. Do not suspend or move the pad with it hanging by its cord.

CAUTION

DO NOT LEAVE IMMOBILE ANIMALS UNATTENDED ON THE PAD. If an animal is immobile, check the pad often to ensure that localized spots are not overheating. If the animal is mobile, allow space for the animal to get on or off the pad at its own discretion.

DO NOT USE A TOWEL between the thermal pad and the patient. It is recommended that any insulating layer such as a towel not be used to cover the pad since any such layer will decrease the heat flow from the pad to the patient. In addition, if the intermediate layer becomes wet with urine or other fluids, the thermal pad sensors may detect a drop in temperature in that area and attempt to compensate for the heat loss with the result that other areas of the pad could overheat. The same situation could occur with fluids on the pad therefore it is recommended that all fluids be wiped up from the pad as quickly as possible.

DO NOT CARRY THE PADS BY THE CORDS.

LIMITED WARRANTY

Each Thermal Controlled Procedure Pad is covered by Prairie Pet Product LLC's (PPP) 3-year limited warranty. Subject to the terms contained herein, all units are warranted to be free of material and workmanship defects for a period of 3 years from the original purchase date.

For three years from the date of its original purchase, PPP will, at its option, repair or replace your product subject to the requirements set forth in this warranty. You will be responsible for shipping costs to return a product for its inspection to determine if the product is entitled to warranty coverage. Any product not meeting the requirements for warranty coverage will be shipped back to you freight collect. PPP will not repair or replace products under warranty that have been damaged due to abuse, damage by animals such as gnawing, or carrying the units by the cord.

During the first year of the warranty period for any product entitled to warranty coverage, PPP will ship the repaired or replaced product at no cost to you.

During the second and third years of warranty coverage, PPP will inform you of a pro-rated cost to repair or replace the unit. This will be equal to the cost of the repair times the number of months from purchase divided by 36. You will be responsible for the costs of shipping the unit to and from PPP. Any such units repaired or replaced will be subject to the warranty assigned to the original unit.

NO OTHER WARRANTY IS EXPRESSED OR IMPLIED. PPC SPECIFICALLY DISCLAIMS THE IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR ANY PARTICULAR PURPOSE. IN THE UNITED STATES, SOME STATES DO NOT ALLOW THE EXCLUSION OF IMPLIED WARRANTIES, SO THE ABOVE EXCLUSION MAY NOT APPLY TO YOU.

EXCLUSIVE REMEDIES

THE REMEDIES PROVIDED HEREIN ARE THE BUYER'S SOLE AND EXCLUSIVE REMEDIES. PPP SHALL NOT BE LIABLE FOR ANY DIRECT, INDIRECT, SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES, WHETHER BASED ON CONTRACT, TORT, OR ANY LEGAL THEORY. IN THE UNITED STATES, SOME STATES DO NOT ALLOW EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGE SO THE LIMITATIONS ABOVE MAY NOT APPLY TO YOU. THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY HAVE OTHER RIGHTS WHICH VARY FROM STATE TO STATE. THIS SECTION DOES NOT LIMIT LIABILITY FOR BODILY INJURY OF A PERSON.

LIMITATION OF DAMAGES

PPP'S ENTIRE LIABILITY FOR ANY DEFECTIVE PRODUCT SHALL IN NO EVENT EXCEED THE PURCHASE PRICE FOR THE DEFECTIVE PRODUCT. THIS LIMITATION APPLIES EVEN IF PPP CANNOT OR DOES NOT REPAIR OR REPLACE ANY DEFECTIVE PRODUCT AND YOUR EXCLUSIVE REMEDY FAILS OF ITS ESSENTIAL PURPOSE.



2948 Kirk Rd., Suite 106
Aurora, IL 60502