



## **POUCH STERILIZATION RACKS**

### Instructions for Use

#### **Intended Usage**

SteriBest Pouch Sterilization Racks are stainless steel racks designed to keep pouches and packs on edge. The racks prevent over stacking (to facilitate sterilant contact) and help pouches to dry after steam sterilization. SteriBest Pouch Sterilization Racks are provided non-sterile. They are compatible with steam, EtO, and gas plasma sterilization.

#### **Preparation**

SteriBest Pouch Sterilization Racks are reusable and must be cleaned using the following cleaning instructions.

#### **Cleaning and Disinfection**

1. Racks should be thoroughly cleaned after each use
2. Used racks should be soaked in enzymatic cleaner that is prepared in accordance with its manufacturer's instructions
3. Used racks can be cleaned using a manual or automated cleaning process
4. Rinse the cleaned racks thoroughly with water
5. After the final rinse, dry the racks

#### **Disposal**

SteriBest Pouch Sterilization Racks have no specific instructions for disposal.

# RICA Surgical Products, Inc.

Phone: 1-800-889-3218 | Fax: 1-847-671-5690 | [www.ricasurgical.com](http://www.ricasurgical.com)  
9207 Ivanhoe Street | Schiller Park, IL 60176